


Now in the clinical lab

BD Veritor™ System analyzes...

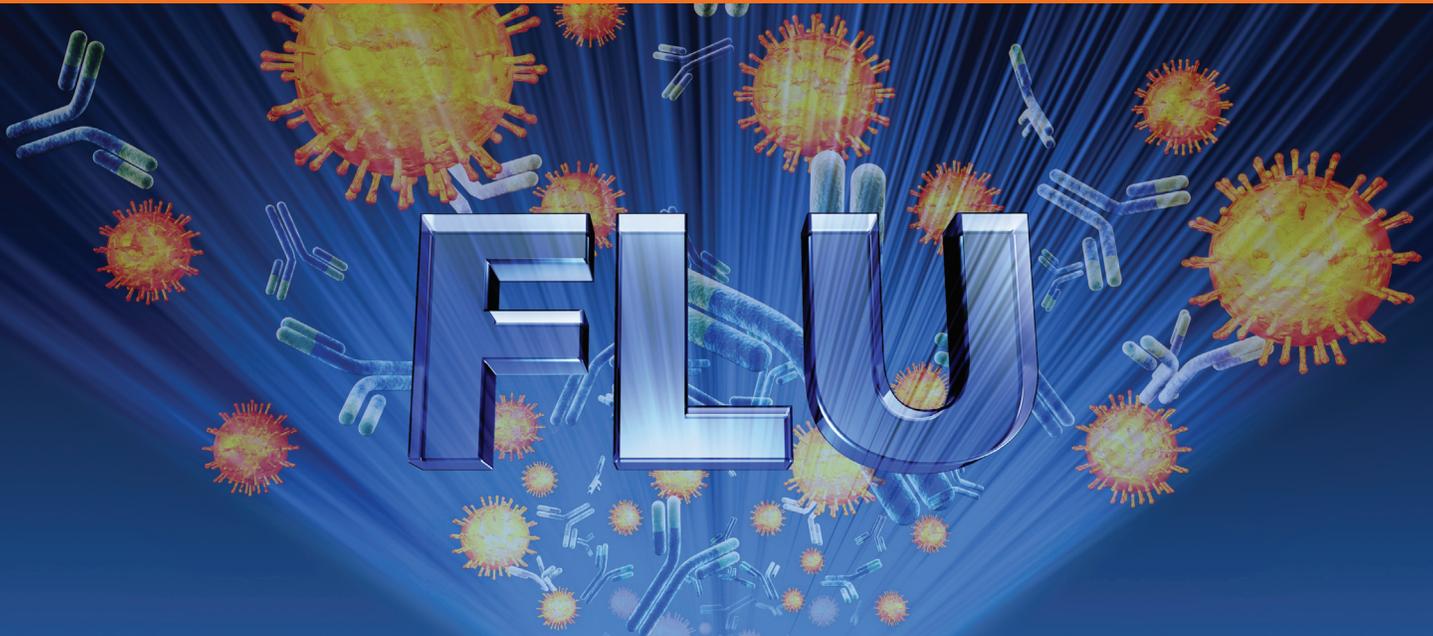

...All that
you see



and all that
you can't.

BD Veritor™ System

For Rapid Detection of Flu A+B



Helping all people
live healthy lives

- **Technology-verified Performance**— Incorporates state of the art technologies that deliver results you can trust
- **Multiple Sample Types**— Validated to be used with nasopharyngeal washes/aspirates, and nasopharyngeal swab in transport
- **Convenient Kit Configuration**— Fully customized kit to test and deliver results in minutes
- **Objective Results**— Easy to read digital display eliminates subjective interpretation

When FLU is suspected...

Detect. Analyze. Verify.

BD Veritor™ System – Proven Performance vs Polymerase Chain Reaction (PCR)

Sample Type	FLU A*		FLU B*	
	PPA	NPA	PPA	NPA
NP Wash/Aspirate – Prospective	83.0%	97.6%	81.3%	99.8%
NP Wash/Aspirate – Retrospective	92.1%	98.9%	74.0%	99.0%
NP Swab in Transport	81.3%	97.4%	85.6%	99.0%

*Data from package insert

State of the Art Technologies

- Advanced Particle Technology enhances sensitivity, helping to improve test performance
- Adaptive Read Technology helps improve specificity to reduce false positive results

Convenient Workflow Delivers Results in Minutes



Easy sample processing
Prefilled unitized tube facilitates workflow



Ready in minutes
Test device is ready to insert into reader 10 minutes after sample is added



Insert and read
Simple one-touch button readies the reader for test device insertion



Objective results
Once the test device is inserted in the reader, an objective, digital test result is displayed in 10 seconds

Ordering Information					
					
Description	Cat. No.	Qty.	Description	Cat. No.	Qty.
BD Veritor™ System Reader	256055	1	BD Veritor™ System Clinical Flu A+B Kit	256041	30



BD Diagnostics
7 Loveton Circle
Sparks, MD 21152-0999
www.bd.com/ds

BD Veritor™ System

Changing the Way You View Rapid Testing

www.bd.com/ds/veritorsystem

BD, BD Logo and BD Veritor are trademarks of Becton, Dickinson and Company
©2012 BD 223561 September 2012
Printed in USA